




**CMC** | **COHMET**  
*let's rehabilitate*




[cohmet-cmc.netlify.app](https://cohmet-cmc.netlify.app)

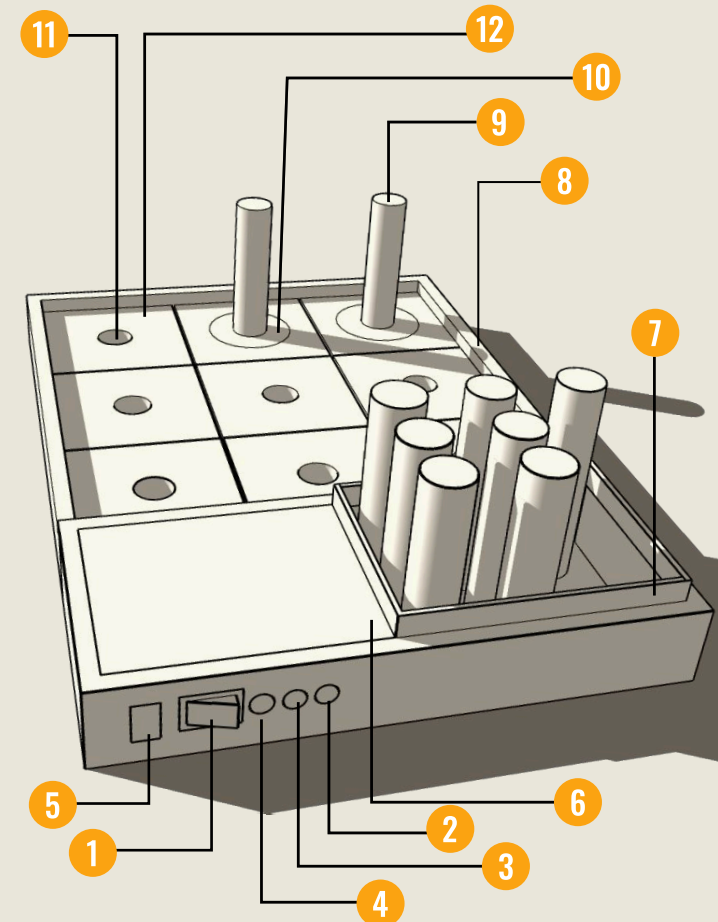

## Know Your Device

- |                        |                        |
|------------------------|------------------------|
| 1. Power button        | 7. Portable peg holder |
| 2. Bluetooth indicator | 8. Wood Casing         |
| 3. Power indicator     | 9. Peg                 |
| 4. Charging indicator  | 10. Well Light         |
| 5. Charging socket     | 11. Well               |
| 6. Circuit holder      | 12. Well tray          |

# STEPS TO USE COHMET

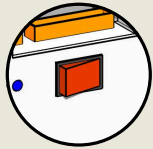

## Step 1

Power ON the device



## Step 2

Open Your App & fill in the details of patient & therapist



## Step 3

Connect the app to the device via Bluetooth (CMC-COHMET)



## Step 4

Click randomize to change the color of wells.




## Step 5

With all set, Click on START to start the game




## Step 6

The wells glow one by one and voice command will start.

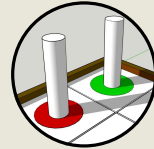


## Step 7

If correct, well turns green  
If wrong, well turns red

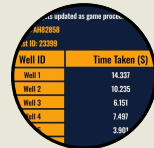


## Step 8

For each well the time taken is updated in the app

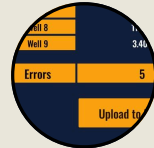

## Step 9

Once all well are done, the total error is updated

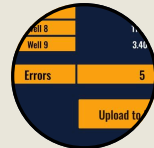


## Step 10

Finally Upload the score to the database.

# USE CASES

- Cognitive Development
- Motor Skills Development
- Rehabilitation Assessment

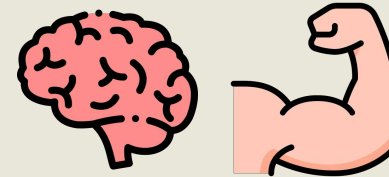

DOWNLOAD  
APP

CMC | COHMET  
*let's rehabilitate*

Proudly Made In INDIA



let's rehabilitate

# FEATURES

- App Controlled
- Portable
- Dynamic Patterns
- Vibrant PEG colors
- Time Tracking
- Error Tracking
- Rechargeable
- Data Protection
- Inbuild Peg Holder
- Cloud Data Storage
- Different Modes
- Modifiable Attachments
- 3 Language support
- 3 LED Indicators
- Color Detection
- Sound Indicator
- Compact Design
- Affordable

designed and developed by Dept Of BIOENGINEERING CMC, VELLORE



# CHARGING AND USE



- Charge the device for **4 Hours** for **8 Hours** of running time.
- Don't charge overnight.
- While charging make sure RED light is on.
- Avoid Playing while charging.